



JOBST

Bella™  
and  
Bella™  
Strong  
Lite

Ready-to-Wear  
**Armsleeves, Gauntlets and Gloves**

# IMPROVED for you

## JOBST® Bella™ Strong

Comfortable, Cost-Effective Compression

### Improved comfort, better compliance, and enhanced quality of life

Patients are more likely to be compliant when they feel comfortable wearing a compression garment. JOBST® Bella™ Strong Ready-to-Wear products – worn together or separately – effectively control moderate to severe lymphedema and edema.

#### JOBST® Bella™ Strong Armsleeve

- Comfort Flat wristband reduces rolling and improves durability
- Available with or without silicone band
- Easy to don, armsleeve comes with donning device
- Available with knitted top band

#### JOBST® Bella™ Strong Gauntlet

- Avoids webbing at thumb area
- Extended wrist avoids pinching at the wrist
- Comfort Flat wrist seam reduces rolling

#### JOBST® Bella™ Strong Glove

- A perfect compliment to the Bella™ Strong Armsleeve
- The Bella™ Strong Gloves offers more sizing options than any comparable ready-to-wear glove in the market
- Utilizes advanced flat-knit technology

### INDICATIONS

Edema • Mild to moderate Lymphedema • Hypertrophic scars

Compression garments can also be used for:

Phlebitis • Post-thrombotic syndrome • Vascular malformations

### CONTRAINDICATIONS


Arterial insufficiency • Cutaneous infection • Dermatitis in the acute phase  
Wet dermatosis • Conditions where increased venous and/or lymphatic return is not desired

# COMFORT for you

## Superior Comfort, Superior Performance

**JOBST® Bella™ Strong** provides patients with the latest development in fiber science. The skin-friendly multifiber nylon combines the clinically effective management of lymphedema with a natural feeling all day long, day after day.

**JOBST® Advanced comfort**

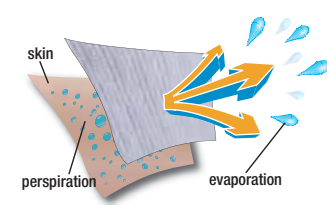

- Quickly wicks moisture
- Keeps you cool and comfortable
- Feels soft to the skin
- Helps to regulate relative humidity, feeling cooler in warm conditions and warmer in cool conditions

In addition, the open knit structure allows for an improved air permeability that keeps patients' skin drier than the competition.



Bella™ Strong: 20-30 mmHg



Bella™ Strong: 30-40 mmHg



Competitor I: 30-40 mmHg



Competitor II: 30-40 mmHg

Silicone dot band to help prevent sliding.



Air permeable, soft and light-weight knit construction.



Additionally, armsleeve garments are available with and without silicone bands for added comfort and fit.

All pictures were taken at equal (50x) magnification.

# INNOVATION for you


## Innovative sizing system - Covers an unrivalled range of patient dimensions

Considering over 50 years of scientific research and practical experience as well as the data from over 36,000 lymphedema patients for the armsleeve and over 33,000 for the gauntlet, JOBST® has developed an innovative measuring-size-system.

Catering to all relevant armsleeve measuring points – wrist, mid-lower and mid-upper arm – this thorough approach, accepts the largest range of patient dimensions. **JOBST® Bella™ Strong** armsleeve sizing begins at the forearm, as data shows that we can accommodate more patients; this allows for multiple size options in the wrist and mid-upper arm for greater coverage, recognizing that every patient is unique.


### JOBST® Bella™ Strong Armsleeve: Ten-size system provides over 80% fit coverage

Competitors' wrist centric size determination limits fit coverage.

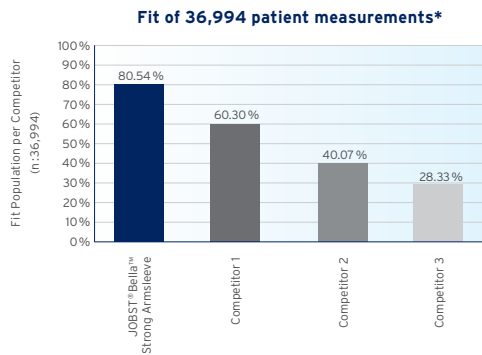

### JOBST® Bella™ Strong Gauntlet: Over 90% fit coverage

With the eight-size system the **JOBST® Bella™ Strong** gauntlet reaches an outstanding range of fit coverage.

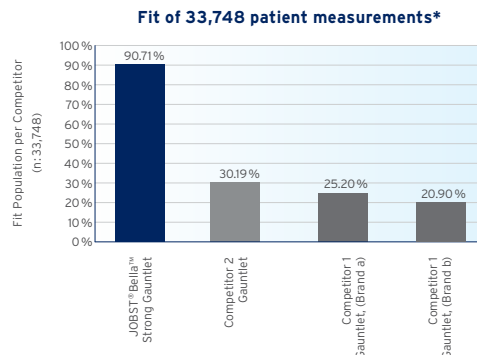


\* Laboratory testing results are intended to illustrate a product's performance under controlled conditions. Actual use results may vary.

## Donning made easy

It requires less force to put on the **JOBST® Bella™ Strong** Ready-to-Wear Armsleeve than those of the competition.

### Competitive Testing: Relative force to don<sup>1</sup>

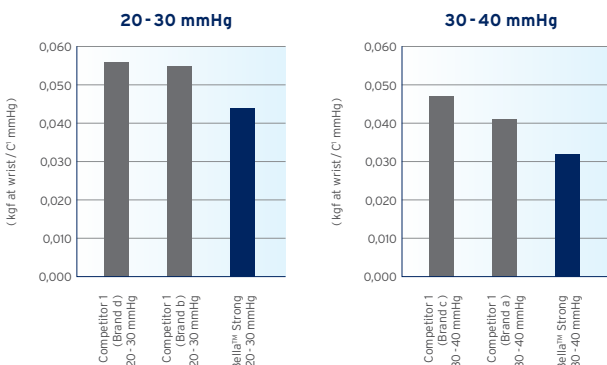

<sup>1</sup> Data on file.

\*Caution: This product contains natural rubber latex which may cause allergic reactions.

### Donning Aids

**JOBST® DONNING AID** included in every armsleeve package.

**JOBST® DONNING GLOVES<sup>\*</sup>** create friction, provide grip and help to distribute the fabric evenly.



# SIZING for you

## JOBST® Bella™ Strong Armsleeve

Size	Wrist (cC)	Mid forearm (cD)	Mid upper (cG)
1	14-18 cm	17-21 cm	22-31 cm
2	14-18 cm	17-21 cm	31-41 cm
3	14-18 cm	21-25 cm	25-34 cm
4	16-19 cm	21-25 cm	28-38 cm
5	17-20 cm	21-25 cm	34-43 cm
6	14-18 cm	25-29 cm	29-38 cm
7	17-20 cm	25-29 cm	33-43 cm
8	18-21 cm	25-29 cm	38-48 cm
9	16-19 cm	29-33 cm	32-42 cm
10	19-22 cm	29-33 cm	36-46 cm

### Sizing instructions

Finding the right size for each patient is easy since stepwise exclusion leads to the perfect solution:

1. Measure wrist circumference.
2. Measure mid forearm circumference.
3. Measure circumference of the mid upper arm.

If you are left with two sizes to choose from, we recommend selecting the larger size for ease of donning, overall comfort, and compliance. Either size selection will deliver the desired therapy.

All sizes are available in 2 lengths:

**Regular** (up to 43 cm); **Long** (greater than 43 cm)



## JOBST® Bella™ Strong Gauntlet

Size	Palm at base of fingers (cA)	Wrist (cC)
1	15-17 cm	14-18 cm
2	17-19 cm	14-18 cm
3	17-19 cm	18-21 cm
4	19-21 cm	14-18 cm
5	19-21 cm	18-21 cm
6	21-23 cm	16-19 cm
7	21-23 cm	19-22 cm
8	23-26 cm	19-22 cm

### Sizing instructions

1. Measure palm circumference.  
This leaves 1 or 2 choices.
2. Measure wrist circumference.
3. Select product.



## JOBST® Bella™ Strong Glove

Size	Palm at base of fingers (cA)	Wrist (cC)
1	16.6 - 18 cm	15.6 - 16.7 cm
2	16.6 - 18 cm	16.7 - 18.1 cm
3	18 - 19.5 cm	15.9 - 17 cm
4	18 - 19.5 cm	17 - 18.5 cm
5	18 - 19.5 cm	18.5 - 20.4 cm
6	19.5 - 21.5 cm	17.2 - 18.7 cm
7	19.5 - 21.5 cm	18.7 - 20.6 cm
8	21.5 - 23.1 cm	18.3 - 20.1 cm

### Sizing instructions

1. Measure palm circumference.  
This leaves 1 or 2 choices.
2. Measure wrist circumference.
3. Select product.



# ORDERING for you



## JOBST® Bella™ Strong Armsleeve

### Compression Classes

- 15 - 20 mmHg
- 20 - 30 mmHg
- 30 - 40 mmHg

### Style



**JOBST Bella™ Strong** Armsleeve is available in ten ready-to-wear sizes and two lengths.

### Colors



Size	15-20 mmHg* NATURAL				20-30 mmHg* NATURAL				30-40 mmHg* NATURAL			
	Silicone Band		No Silicone		Silicone Band		No Silicone		Silicone Band		No Silicone	
	Regular	Long	Regular	Long	Regular	Long	Regular	Long	Regular	Long	Regular	Long
1	102251	102271	102261	102281	102331	102351	102341	102361	102411	102431	102421	102441
2	102252	102272	102262	102282	102332	102352	102342	102362	102412	102432	102422	102442
3	102253	102273	102263	102283	102333	102353	102343	102363	102413	102433	102423	102443
4	102254	102274	102264	102284	102334	102354	102344	102364	102414	102434	102424	102444
5	102255	102275	102265	102285	102335	102355	102345	102365	102415	102435	102425	102445
6	102256	102276	102266	102286	102336	102356	102346	102366	102416	102436	102426	102446
7	102257	102277	102267	102287	102337	102357	102347	102367	102417	102437	102427	102447
8	102258	102278	102268	102288	102338	102358	102348	102368	102418	102438	102428	102448
9	102259	102279	102269	102289	102339	102359	102349	102369	102419	102439	102429	102449
10	102260	102280	102270	102290	102340	102360	102350	102370	102420	102440	102430	102450

Please refer to your Price List for a complete list of catalog numbers for ordering.

## JOBST® Bella™ Strong Gauntlet

### Compression Classes

- 15 - 20 mmHg
- 20 - 30 mmHg

### Style



**JOBST Bella™ Strong** Gauntlet is available in eight ready-to-wear sizes.

### Colors



Size	15-20 mmHg**		20-30 mmHg**	
	Natural	Black	Natural	Black
1	101661	101653	101677	101669
2	101662	101654	101678	101670
3	101663	101655	101679	101671
4	101664	101656	101680	101672
5	101665	101657	101681	101673
6	101666	101658	101682	101674
7	101667	101659	101683	101675
8	101668	101660	101684	101676



\* Mean compression for average forearm size.  
\*\* Mean compression for an average palm size.



# ORDERING for you

## JOBST® Bella™ Strong Glove

The Bella™ Strong glove is the next evolution in lymphatic ready-to-wear market. Leveraging the advanced flat-knitting technology seen in our custom garments and offered in eight sizes, the Bella™ Strong glove gives patients a nearly custom feel at a ready-to-wear price.

### Compression Classes

15-20 mmHg

20-30 mmHg

### Style



**JOBST Bella™ Strong**  
Glove is available in eight  
ready-to-wear sizes.

### Colors



natural

black



Optional: the Bella™ Strong  
Glove can also be worn  
with the Elvarex® custom  
armsleeve.

	15-20 mmHg**		20-30 mmHg**	
Size	Natural	Black	Natural	Black
1	7520040	7520030	7529140	7529130
2	7520041	7520031	7529141	7529131
3	7520042	7520032	7529142	7529132
4	7520043	7520033	7529143	7529133
5	7520044	7520034	7529144	7529134
6	7520045	7520035	7529145	7529135
7	7520046	7520036	7529146	7529136
8	7520047	7520037	7529147	7529137

# IMPROVED for you

## JOBST® Bella™ Lite

### The Entry Point to JOBST® Quality

The **JOBST® Bella™ Lite** Ready-to-Wear Armsleeve, Gauntlet and Glove, together or separately, help maintain the limb size of lymphedema patients. These comfortable garments provide gradient compression to assist in patient management following trauma, surgery or radiation therapy.



#### FEATURES AND BENEFITS

- Soft and air permeable
- Lightweight, knit construction
- Fits a wide range of arm and hand shapes
- Reduced compression at the wrist provides an effective and comfortable fit when the Armsleeve and Gauntlet are worn together
- Easy to put on, comfortable to wear
- Durable
- Armsleeve available with silicone dot band

#### INDICATIONS

Edema • Lymphedema

Compression garments can also be used for:

Phlebitis • Post-thrombotic syndrome • Vascular malformations • Post-liposuction<sup>†</sup>

#### CONTRAINDICATIONS

Arterial insufficiency • Cutaneous infection • Dermatitis in the acute phase  
Wet dermatosis • Conditions where increased venous and/or lymphatic return is not desired


\* The mean compression for an average forearm size. \*\* The mean compression for an average palm size. † 20-30 mmHg Armsleeve only.



# SIZING for you


## JOBST® Bella™ Lite Armsleeve

Size	Wrist	Mid-Lower	Mid-Upper
S	13.5 cm - 19.5 cm	17.8 cm - 23.2 cm	21.5 cm - 39.5 cm
M	15.5 cm - 21.5 cm	23.3 cm - 28.7 cm	27 cm - 45 cm
L	17.5 cm - 23.5 cm	28.8 cm - 34.3 cm	32.5 cm - 50.5 cm

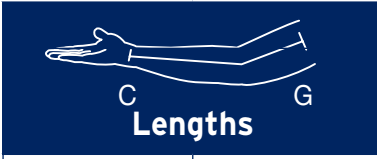


Lengths	
Regular (up to 43 cm)	Long (over 43 cm)

## JOBST® Bella™ Lite Gauntlet

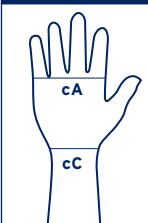

Size	Palm at base of fingers (cA)	Wrist (cC)
S	15 cm - 19 cm	14 cm - 20 cm
M	19 cm - 22 cm	16 cm - 22 cm
L	22 cm - 26 cm	18 cm - 24 cm

## JOBST® Bella™ Lite Glove

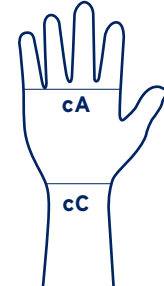Size	(cA)	(cC)
1	16.6 - 18 cm	15.6 - 16.7 cm
2	16.6 - 18 cm	16.7 - 18.1 cm
3	18 - 19.5 cm	15.9 - 17 cm
4	18 - 19.5 cm	17 - 18.5 cm
5	18 - 19.5 cm	18.5 - 20.4 cm
6	19.5 - 21.5 cm	17.2 - 18.7 cm
7	19.5 - 21.5 cm	18.7 - 20.6 cm
8	21.5 - 23.1 cm	18.3 - 20.1 cm

# ORDERING for you

## JOBST® Bella™ Lite Ready-to-Wear Armsleeve



### Compression Classes

15 - 20 mmHg

20 - 30 mmHg

### Style



**JOBST Bella™ Lite** Armsleeve is available in three ready-to-wear sizes and two lengths.

### Colors



beige brown black

Size	Length	Beige	Brown	Black	mmHg
S	Regular	101410			15-20 mmHg
	Long	101328			
M	Regular	101411			15-20 mmHg
	Long	101329			
L	Regular	101412			15-20 mmHg
	Long	101330			
S/Silicone dot band	Regular	101413			15-20 mmHg
	Long	101331			
M/Silicone dot band	Regular	101414			15-20 mmHg
	Long	101332			
L/Silicone dot band	Regular	101415			15-20 mmHg
	Long	101333			
S	Regular	101416			20-30 mmHg
	Long	101313	101337	101340	
M	Regular	101417			20-30 mmHg
	Long	101314	101338	101341	
L	Regular	101418			20-30 mmHg
	Long	101315	101339	101342	
S/Silicone dot band	Regular	101419			20-30 mmHg
	Long	101334			
M/Silicone dot band	Regular	101420			20-30 mmHg
	Long	101335			
L/Silicone dot band	Regular	101421			20-30 mmHg
	Long	101336			

## JOBST® Bella™ Lite Ready-to-Wear Gauntlet



Size	Beige	Brown	Black	mmHg
S	101325			15-20 mmHg
M	101326			15-20 mmHg
L	101327			15-20 mmHg
S	101319	101355	101358	20-30 mmHg
M	101320	101356	101359	20-30 mmHg
L	101321	101357	101360	20-30 mmHg

### Compression Classes

15 - 20 mmHg

20 - 30 mmHg

### Style



**JOBST Bella™ Lite** Gauntlet is available in three ready-to-wear sizes.

### Colors



beige brown black

# ORDERING for you

## JOBST® Bella™ Lite Glove

The **compression glove** that goes **hand-in-hand** with other JOBST® Bella Lite products

Durable and latex-free, the JOBST Bella Lite glove provides gentle compression for patients with mild to moderate edema. Its ultra-thin fabric and fine seams ensure great wearing comfort. The glove is available in **eight sizes**, covering a broad range of patients and provides a perfect complement to the rest of the JOBST Bella Lite assortment.

### Donning the glove

The glove can be worn on either hand: On the right hand, the label is visible on the wrist. To wear on the left hand, turn the glove inside out, inverting the fingers. The label is then on the inside.

#### Follow these steps when putting on your glove:

1. Slip fingers into glove and position it comfortably around the palm and thumb (Fig. 1).
2. The glove should be positioned so that the fingers end near the beds of the fingernails (Fig. 2).
3. If the fingers are too long, mark the required length, remove the glove and trim accordingly.



Fig. 1



Fig. 2

**Take care not to remove too much of the finger length. Open wounds should be covered before applying glove. Remove jewelry before applying.**

### Compression Classes

15 - 20 mmHg

20 - 30 mmHg

### Colors



black



beige

### Ordering Information

Size	Beige	Black	mmHg
1	7769700	7769710	15-20 mmHg
2	7769701	7769711	15-20 mmHg
3	7769702	7769712	15-20 mmHg
4	7769703	7769713	15-20 mmHg
5	7769704	7769714	15-20 mmHg
6	7769705	7769715	15-20 mmHg
7	7769706	7769716	15-20 mmHg
8	7769707	7769717	15-20 mmHg
1	7769800	7769810	20-30 mmHg
2	7769801	7769811	20-30 mmHg
3	7769802	7769812	20-30 mmHg
4	7769803	7769813	20-30 mmHg
5	7769804	7769814	20-30 mmHg
6	7769805	7769815	20-30 mmHg
7	7769806	7769816	20-30 mmHg
8	7769807	7769817	20-30 mmHg

### Style



**JOBST Bella™ Lite Glove** is available in eight ready-to-wear sizes.

# JOBST® Custom Flat-Knit Garments for Upper Extremities

## JOBST® Elvarex® Custom Armsleeve

CCL	Compression mmHg	Beige	Black	Honey	Cranberry	Caramel	Cherry	Navy
1	15-21 mmHg	●	●	●	●	●	●	●
2	23-32 mmHg	●	●	●	●	●	●	●
2 Forte	23-32 mmHg	●	●	●	●	●	●	●

## JOBST® Elvarex® Custom Glove/Gauntlet

CCL	Compression mmHg	Beige	Black	Honey	Cranberry	Caramel	Cherry	Navy
1	15-21 mmHg	●	●	●	●	●	●	●
2	23-32 mmHg	●	●	●	●	●	●	●
2 Forte	23-32 mmHg	●	●	●	●	●	●	●

## JOBST® Elvarex® Soft Custom Armsleeve

CCL	Compression mmHg	Beige	Black	Cocoa	Honey	Cranberry	Grey	Cherry	Navy
1	15-21 mmHg	●	●	●	●	●	●	●	●
2	23-32 mmHg	●	●	●	●	●	●	●	●

## JOBST® Elvarex® Soft Seamless Custom Glove / Gauntlet

CCL	Compression mmHg	Beige	Black	Cocoa	Honey	Cranberry	Grey	Cherry	Navy
1	15-21 mmHg	●	●	●	●	●	●	●	●
2	23-32 mmHg	●	●	●	●	●	●	●	●

Please refer to the product label and/or package insert for full instructions on the safe use of these products.  
All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (glove / gauntlet).



JOBST®,  
an Essity brand



/JOBSTUSA



@JOBST\_USA



@JOBSTforUSA



jobst-usa.com

BSN medical Inc., an Essity company  
5825 Carnegie Blvd., Charlotte, NC 28209-4633  
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325  
To order toll-free: Tel. (+1) 800 537 1063

61674 R1 ©2018 BSN medical Inc. REV 12/18