

Ready-to-Wear

Armsleeves and Gauntlets

JOBST® Bella™ Strong

Comfortable Compression

Improved comfort, better compliance, and enhanced quality of life

Patients are more likely to be compliant when they feel comfortable wearing a compression garment. **JOBST® Bella™ Strong Ready-to-Wear** products – worn together or separately – effectively control for moderate lymphedema and edema.

JOBST® Bella™Strong Armsleeve

- · Comfort Flat wristband reduces rolling and improves durability
- · Available with or without silicone band
- Easy to don, armsleeve comes with donning device
- · Available with knitted top band

JOBST® Bella™ Strong Gauntlet

- · Avoids webbing at thumb area
- Extended wrist avoids pinching at the wrist
- · Comfort Flat wrist seam reduces rolling

JOBST® Bella™ Lite

The Entry Point to JOBST® Quality

The **JOBST**® **Bella**™ **Lite** Ready-to-Wear Armsleeve and Gauntlet, together or separately, help maintain the limb size of lymphedema patients. These comfortable garments provide gradient compression to assist in patient management following trauma, surgery or radiation therapy.

- Soft and air permeable, lightweight, durable knit construction
- Reduced compression at the wrist provides an effective and comfortable fit when the Armsleeve and Gauntlet are worn together
- Easy to put on, comfortable to wear
- Armsleeve available with silicone dot band



INDICATIONS JOBST® BELLA™ STRONG

Armsleeve Compression garments can also be used for: Lymphedema and other edema

Compression garments can also be used for:

Phlebitis • Post-thrombotic syndrome • Vascular malformations

INDICATIONS JOBST® BELLA™ LITE

Armsleeve Compression garments can also be used for: Lymphedema and other edema

Compression garments can also be used for:

Phlebitis • Post-thrombotic syndrome • Vascular malformations • Post-liposuction[†]

CONTRAINDICATIONS

Arterial insufficiency • Untreated septic phlebitis
Conditions where increased venous and/or lymphatic return is not desired

COMFORT for you

Comfort and Performance

JOBST® Bella™ Strong provides patients with the latest development in fiber science. The skin-friendly multifiber nylon combines the clinically effective management of lymphedema with a natural feeling all day long, day after day.





humidity, feeling cooler in warm conditions and warmer in cool

conditions

In addition, the open knit structure allows for an improved air permeability that keeps patients' skin drier than the competition.



Bella™ Strong: 20-30 mmHg



Bella™ Strong: 30-40 mmHg



Competitor I: 30-40 mmHg



Competitor II: 30-40 mmHg

INNOVATION for you


Innovative sizing system - Covers an unrivalled range of patient dimensions

Considering over 50 years of scientific research and practical experience as well as the data from over 36,000 lymphedema patients for the armsleeve and over 33,000 for the gauntlet, JOBST® has developed an innovative measuring-size-system.

Catering to all relevant armsleeve measuring points – wrist, mid-lower and mid-upper arm – this thorough approach, accepts the largest range of patient dimensions. **JOBST® Bella™ Strong** armsleeve sizing begins at the forearm, as data shows that we can accommodate more patients; this allows for multiple size options in the wrist and mid-upper arm for greater coverage, recognizing that every patient is unique.


JOBST® Bella™ Strong Armsleeve: Ten-size system provides over 80% fit coverage

Competitors' wrist centric size determination limits fit coverage.

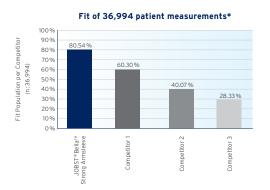

JOBST® Bella™ Strong Gauntlet: Over 90% fit coverage

With the eight-size system the **JOBST® Bella™ Strong** quuntlet reaches an outstanding range of fit coverage.

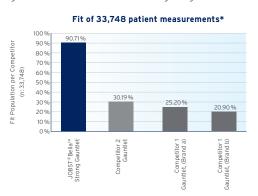


^{*} Laboratory testing results are intended to illustrate a product's performance under controlled conditions. Actual use results may vary.

Donning made easy

It requires less force to put on the **JOBST® Bella™ Strong** Ready-to-Wear Armsleeve than those of the competition.

Competitive Testing: Relative force to don¹

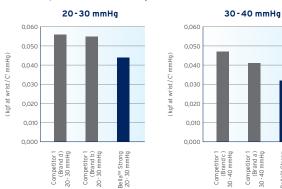

Donning Aids

JOBST® DONNING AID included in every armsleeve package.

JOBST® DONNING GLOVES* create friction, provide grip and help to distribute the fabric evenly.





¹ Data on file

^{*}Caution: This product contains natural rubber latex which may cause allergic reactions.

SIZING for you

JOBST® Bella™ Strong Armsleeve

Size	Wrist (cC)	Mid forearm (cD)	Mid upper (cG)
1	14-18 cm	17-21 cm	22-31 cm
2	14-18 cm	17-21 cm	31-41 cm
3	14-18 cm	21-25 cm	25-34 cm
4	16-19 cm	21-25 cm	28-38 cm
5	17 - 20 cm	21-25 cm	34-43 cm
6	14-18 cm	25-29 cm	29-38 cm
7	17 - 20 cm	25-29 cm	33-43 cm
8	18-21 cm	25-29 cm	38-48 cm
9	16-19 cm	29-33 cm 32-42 cm	
10	19-22 cm	29-33 cm	36-46 cm /

Sizing instructions

Finding the right size for each patient is easy since stepwise exclusion leads to the perfect solution:

- 1. Measure wrist circumference.
- 2. Measure mid forearm circumference.
- 3. Measure circumference of the mid upper arm.

If you are left with two sizes to choose from, we recommend selecting the larger size for ease of donning, overall comfort, and compliance. Either size selection will deliver the desired therapy.

JOBST® Bella™ Strong Gauntlet

	Size	Palm at base of fingers (cA)	Wrist (cC)
٥٥٥	1	15-17 cm	14-18 cm
	2	17 - 19 cm	14-18 cm
CA CA	3	17 - 19 cm	18 - 21 cm
	4	19 - 21 cm	14-18 cm
	5	19 - 21 cm	18 - 21 cm
cC (6	21-23 cm	16-19 cm
	7	21-23 cm	19-22 cm
1	8	23-26 cm	19-22 cm


Sizing instructions

- 1. Measure palm circumference. This leaves 1 or 2 choices.
- 2. Measure wrist circumference.
- 3. Select product.



JOBST® Bella™ Lite Armsleeve

Size	Wrist	Mid-Lower	Mid-Upper
S	13.5 cm - 19.5 cm	17.8 cm - 23.2 cm	21.5 cm - 39.5 cm
М	15.5 cm - 21.5 cm	23.3 cm-28.7 cm	27 cm - 45 cm
L	17.5 cm - 23.5 cm	28.8 cm-34.3 cm	32.5 cm - 50.5 cm

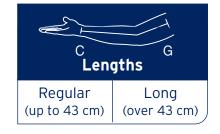

All sizes are available in 2 lengths:

Regular (up to 43 cm); Long (greater than 43 cm)



JOBST® Bella™ Lite Gauntlet

	Size	Palm at base of fingers (cA) Wrist (cC)	
S M	S	15 cm - 19 cm	14 cm - 20 cm
	М	19 cm -22 cm	16 cm-22 cm
cC (L	22 cm - 26cm	18 cm - 24 cm

ORDERING for you

JOBST® Bella™ Strong Armsleeve

Compression Classes

Style

Colour



20-30 mmHg

JOBST Bella™ Strong Armsleeve is available in ten ready-to-wear sizes.



30-40 mmHg

	15-20 r NATU		20-30 mmHg* NATURAL		30-40 mmHg* NATURAL	
Size	Silicone Band	No Silicone	Silicone Band	No Silicone	Silicone Band	No Silicone
1	7272701	7272711	7273601	7273611	7277401	7277411
2	7272702	7272712	7273602	7273612	7277402	7277412
3	7272703	7272713	7273603	7273613	7277403	7277413
4	7272704	7272714	7273604	7273614	7277404	7277414
5	7272705	7272715	7273605	7273615	7277405	7277415
6	7272706	7272716	7273606	7273616	7277406	7277416
7	7272707	7272717	7273607	7273617	7277407	7277417
8	7272708	7272718	7273608	7273618	7277408	7277418
9	7272709	7272719	7273609	7273619	7277409	7277419
10	7272710	7272720	7273610	7273620	7277410	7277420

Please refer to your Price List for a complete list of catalogue numbers for ordering.

JOBST® Bella™ Strong Gauntlet

Compression Classes

15-20 mmHg

20-30 mmHg

Style



JOBST Bella™ Strong Gauntlet is available in eight ready-to-wear sizes. Colour



	15-20 mmHg**	20-30 mmHg**
Size	Natural	Natural
1	7277700	7369440
2	7277701	7369441
3	7277702	7369442
4	7277703	7369443
5	7277704	7369444
6	7277705	7369445
7	7277706	7369446
8	7277707	7369447



^{*} Mean compression for average forearm size.

^{**} Mean compression for an average palm size.

ORDERING for you

JOBST® Bella™ Lite Ready-to-Wear Armsleeve



Compression Classes

15-20 mmHg

20-30 mmHg

Style



JOBST Bella™ Lite Armsleeve is available in three ready-towear sizes and two lengths.

Colours





Size	Length	Beige	Black	mmHg
C	Regular	7527302		15 30 mm l a
S	Long	101328		15-20 mmHg
M	Regular	7527402		15-20 mmHg
IVI	Long	101329		15-20 111111111111111111111111111111111111
L	Regular	7527502		15-20 mmHg
L	Long	101330		15 20 11111119
S/Silicone	Regular	7527303		15-20 mmHg
dot band	Long	101331		13 20 11111119
M/Silicone	Regular	7527403		15-20 mmHg
dot band	Long	101332		13 20 11111119
L/Silicone	Regular	7527503		15-20 mmHg
dot band	Long	101333		13 20 11111119
S	Regular	7527605		20-30 mmHg
	Long	101313	101340	20 00 mm rg
M	Regular	7527705		20-30 mmHg
171	Long	101314	101341	20 30 11111119
L	Regular	7527805		20-30 mmHg
	Long	101315	101342	20 30 11111119
S/Silicone	Regular	7527606		20-30 mmHg
dot band	Long	101334		20 30 11111119
M/Silicone	Regular	7527706		20-30 mmHg
dot band	Long	101335		20 00 11111119
L/Silicone	Regular	7527806		20-30 mmHg
dot band	Long	101336		20 30 11111119

JOBST® Bella™ Lite Ready-to-Wear Gauntlet



Size	Beige	Black	mmHg
S	101325		15-20 mmHg
М	101326		15-20 mmHg
L	101327		15-20 mmHg
S	101319	101358	20-30 mmHg
М	101320	101359	20-30 mmHg
L	101321	101360	20-30 mmHg

Compression Classes

15-20 mmHg 20-30 mmHg Style



Colours







JOBST® Custom Flat-Knit Garments for Upper Extremities

When Ready-to-Wear sizing is not a perfect fit, customized garments are available through Elvarex®

JOBST® Elvarex® Custom Armsleeve / Glove / Gauntlet

CCL	Compression mmHg	Beige	Black	Navy	Honey	Cranberry	Caramel
1	15 - 21 mmHg		•	•		•	•
2	23-32 mmHg		•	•		•	•
2 Forte	23-32 mmHg		•	•		•	

JOBST® Elvarex® Plus Custom Armsleeve / Glove / Gauntlet

CCL	Compression mmHg	Beige	Black	Navy	Honey	Cranberry	Caramel
1	15 - 21 mmHg		•	•		•	
2	23-32 mmHg		•	•		•	
2 Forte	23-32 mmHg		•	•		•	

JOBST® Elvarex® Soft Custom Armsleeve

CCL	Compression mmHg	Beige	Black	Navy	Honey	Cranberry
1	15 - 21 mmHg		•	•		•
2	23-32 mmHg		•	•		

JOBST® Elvarex® Soft Seamless Custom Glove / Gauntlet

CCL	Compression mmHg	Beige	Black	Navy	Honey	Cranberry
1	15 - 21 mmHg		•	•		•
2	23-32 mmHg		•	•		

Please refer to the product label and/or package insert for full instructions on the safe use of these products. All compressions shown are the mean compression for an average forearm size (armsleeve) or mean compression for an average palm size (glove / gauntlet).



