









Product Positioning - Upper Extremity Edema / Lymphedema



Product	Basic Styles	Compression Level	Suggestions for Use According to Severity From _____ To _____ <small>Listed according to severity of condition, with the most appropriate products at bottom for Stage III - Severe</small>	
JOBST® Bella™ Lite Ready-To-Wear Circular Knit	 <p>Armsleeve Glove Gauntlet Comb. Armsleeve and Gauntlet</p>	15-20mmHg* 20-30mmHg*	Stage 0 - Latency At Risk, No Swelling Normal Tissue Density Early Intervention Exercising, Traveling	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes
JOBST® Bella™ Strong Ready-To-Wear Circular Knit	 <p>Armsleeve Glove Gauntlet</p>	15-20mmHg* 20-30mmHg* 30-40mmHg* <small>(not available in the Glove)</small>	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Better Containment	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting) Better Containment
JOBST® Confidence Custom-made Flat-Knit	 <p>Armsleeve</p>	Class 1 (15-21mmHg)** Class 2 (23-32mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Active Lifestyle Fragile Skin Lipedema / Lymphedema	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting, no skin folds) Lipedema / Lymphedema
Elvarex® Soft Custom-made Flat Knit	 <p>Armsleeve Glove Gauntlet</p>	Class 1 (15-21mmHg)** Class 2 (23-32mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting)
Elvarex® Soft Seamless Custom-made Flat Knit	 <p>Glove Gauntlet</p>	Class 1 (15-21mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting)
Elvarex®*** † Custom-made Flat Knit Rubber Core	 <p>Armsleeve Glove Gauntlet</p>	Class 1 (15-21mmHg)** Class 2 (23-32mmHg)** Class 2 forte (23-32mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Bridges Folds	Stage III - Severe Irreversible, Elephantiasis Extreme increase in volume Tissue & Texture changes High Containment
Elvarex® Plus Custom-made Flat 3D Knit	 <p>Glove Gauntlet</p>	Class 1 (15-21mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Bridges Folds	Stage III - Severe Irreversible, Elephantiasis Extreme increase in volume Tissue & Texture changes High Containment
Custom Seamed*** Custom-made Cut and Sew	 <p>Armsleeve Glove Gauntlet</p>	20-90+mmHg**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Deformities	Stage III - Severe Irreversible, Elephantiasis Extreme increase in volume Tissue & Texture changes Deformities

* The mean compression for an average forearm. ** Design Pressure. *** NOTE: The suggested use for this product is from Stage I - Mild to Stage III - Severe. This includes use for Stage II - Moderate
 † Caution: This product contains natural rubber latex which may cause allergic reactions.

Product Positioning - Lower Extremity Edema / Lymphedema



Product	Basic Styles	Compression Level	Suggestions for Use According to Severity	
			From	To
JOBST® Relief®*** Ready-To-Wear Circular Knit		15-20mmHg* 20-30mmHg* 30-40mmHg*	Stage 0 - Latency At Risk, No Swelling Normal Tissue Density Early Intervention Exercising, Traveling	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting)
Seamless Soft Custom-made Circular Knit		18-21mmHg* 23-32mmHg* 34-46mmHg*	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting)
Bellavar™ Custom-made Circular Knit		23-32mmHg* 34-46mmHg*	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Conforming Fit	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting) Strong Containment
JOBST® Confidence Custom-made Flat-Knit		Class 1 (15-21mmHg)** Class 2 (23-32mmHg)** Class 3 (34-46mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Active Lifestyle Fragile Skin Lipedema / Lymphedema	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting, no skin folds) Lipedema / Lymphedema
Elvarex® Soft Custom-made Flat Knit		Class 1 (18-21mmHg)** Class 2 (23-32mmHg)** Class 3 (34-46mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting)
Elvarex® Soft Seamless Custom-made Flat Knit		Class 1 (18-21mmHg)** Class 2 (23-32mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage II - Moderate Spontaneously Irreversible Hardening of Tissue (no pitting)
Elvarex® Custom-made Flat Knit Rubber Core		Class 1 (18-21mmHg)** Class 2 (23-32mmHg)** Class 3 (34-46mmHg)** Class 3 Forte (34-46mmHg)** Class 4 (49-70mmHg)** Class 4 Super (60-90mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage III - Severe Irreversible, Elephantiasis Extreme increase in volume Tissue & Texture changes
Elvarex® Plus Custom-made Seamless, Flat 3D Knit		Class 1 (18-21mmHg)** Class 2 (23-32mmHg)**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes	Stage III - Severe Irreversible, Elephantiasis Extreme increase in volume Tissue & Texture changes
Custom Seamed*** Custom-made Cut and Sew		20-90+mmHg**	Stage I - Mild Reversible / Reduction with Elevation Heaviness / Soft Pitting Edema No Secondary Tissue Changes Deformities	Stage III - Severe Irreversible, Elephantiasis Extreme increase in volume Tissue & Texture changes Deformities

* The mean compression for an average ankle size. ** Design Pressure. *** NOTE: The suggested use for this product is from Stage I - Mild to Stage III - Severe. This includes use for Stage II - Moderate



JOBST®,
an Essity brand



/JOBSTUSA



@JOBST_USA



@JOBSTforUSA



jobst-usa.com

BSN Medical Inc., an Essity company
5825 Carnegie Blvd., Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325
To order toll-free: Tel. (+1) 800 537 1063